# **ANIMAL TOOL CARE**

Cleaning your Wahl clippers blades



### **EVERYDAY CLEANING**





Your tools should be cleaned and sanitized after every client to prevent the spread of infectious skins diseases.



Start by brushing off excess hair on your blades.



Spray your clipper/trimmer blades with Wahl Clini-Clip, and allow to sit before wiping off any excess with a clean towel.



Ensure you also spray your combs, brushes and attachment combs with Clini-Clip as well.



Make sure you oil your blades immediately after using Clini-Clip with Wahl Blade Oil.

#### **ONCE A MONTH CLEANING**



testing materials or grinding compound may be found on the blades. This is easily cleaned off using the enclosed blade

# **ANIMAL TOOL CARE**

Oiling your Wahl clippers blades



### **HOW TO OIL YOUR BLADES**





With the blade directed downwards, turn the clipper/trimmer on.



Apply three drops of oil – one on each side and one in the middle.



With the clipper/trimmer still on, move the taper lever back and forth to work oil into blade.



Turn off clipper/trimmer and blot excess oil with clean towel.

Always use Wahl Clipper Oil on your Wahl tools as it is specifically designed to extend the life of your blades.